Pulse O_x

Quick Start Guide

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Thank you for choosing the Withings Pulse O_x

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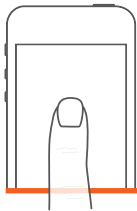
➔ support.withings.com/pulse

Quick start from your Smartphone or Tablet

FR Démarrage rapide depuis votre Smartphone | **DE** Schnellstart von Ihrem Smartphone | **ES** Inicio rápido desde tu Smartphone | **IT** Avvio rapido dallo Smartphone | **PT** Iniciação rápida a partir de seu Smartphone

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
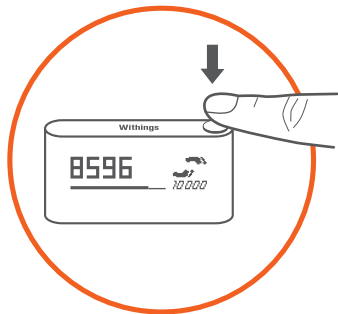
Personalize your Pulse O, screens directly from the App, choosing what matters most to you.

Analyze your sleep to improve its quality

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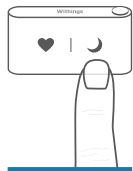
2

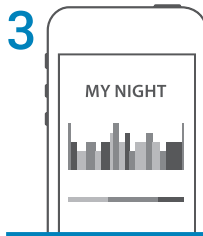
3


Measure your **heart rate** & **blood oxygen level**

FR Mesurer votre fréquence cardiaque et niveau d'oxygénation du sang | DE Messen Sie Ihre Herzfrequenz und die Sauerstoffsättigung des Blutes | ES Mide tu frecuencia cardiaca y tu nivel de oxígeno en sangre | IT Misura la frequenza cardiaca e il livello di ossigenazione del sangue | PT Meça sua frequência cardíaca e nível da oxigenação do sangue

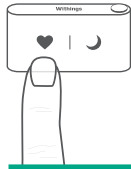1

2

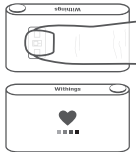Place your finger gently on the light sensor on the back.

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
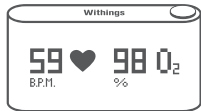
3

Constant activity updates to your Smartphone

FR Synchronisation automatique avec votre smartphone | DE Automatische Synchronisation mit Ihrem Smartphone
ES Tu actividad constantemente sincronizada con tu smartphone | IT Sincronizzazione automatica dal tuo smartphone
PT Sincronização automática com seu smartphone

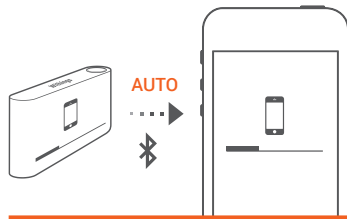i **Press the top button for 3 seconds to start a manual synchronization.**
FR Appuyer sur le bouton du haut pendant 3s pour lancer une synchronisation manuelle.
DE Drücken Sie die Taste oben 3 Sekunden lang, um eine manuelle Synchronisierung zu starten.
ES Pulsa el botón superior durante 3 segundos para comenzar la sincronización manual.
IT Premi il tasto in alto per 3 sec. per lanciare la sincronizzazione manuale.
PT Pressione o botão superior durante 3s para iniciar a sincronização manual.

Plug your Pulse to computer to charge it, not to synchronise your data.
FR Câble pour charger et non pour la synchro des données.
DE Ladekabel. Keine Daten-Synchronisierung.
ES Conéctalo al ordenador para recargar la batería, no para sincronizar los datos.
IT Cavo per ricarica e non per sincronizzazione dati.
PT Ligue o Pulse ao computador para carregar, não para sincronizar dados.

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
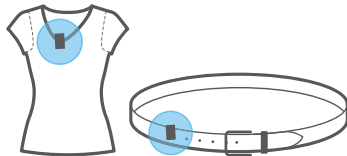
Warning

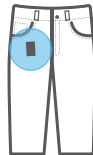If you wear your tracker with the clip we recommend clipping it toward the inside of your pocket for maximum security.

In your Pocket

FR Dans votre poche
DE In der Tasche
ES En el bolsillo
IT Nella tasca
PT Em seu bolso

Discover your trends with the online free tool

FR Découvrez vos tendances avec l'application mobile et l'outil gratuit en ligne
DE Mit der Handy-App und dem kostenlosen Online-Tool erkennen Sie Ihre Trends | **ES** Aprende más sobre tus tendencias con la aplicación móvil y la herramienta en línea gratuita | **IT** Scopri le tue tendenze con l'app mobile e lo strumento gratuito online | **PT** Descubra suas tendências com o aplicativo móvel e a ferramenta gratuita online

➔ healthmate.withings.com

FCC ID: XNAWAM01

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

This device complies with the R&TTE Directive.

A copy of the EU Declaration of Conformity is available online
at www.withings.com/compliance

This product is not a medical device and should not be used
to diagnose or treat any medical condition.

Withings Pulse O_x | August 27th, 2014
©2014 Withings. All right reserved.

withings.com/pulse